


BEFORE YOU BEGIN

Bring vials to room temperature. Wash hands and put on gloves.
Remove vial caps. Cleanse stoppers with alcohol swab and allow to dry.


White = Product → 
Clear = Water → 

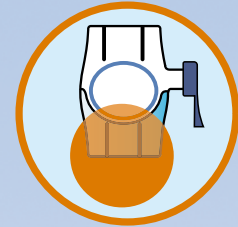


1 CONNECT water vial



2 CONNECT product vial

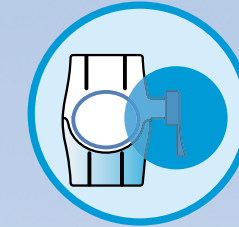


3 WITHDRAW mixed product

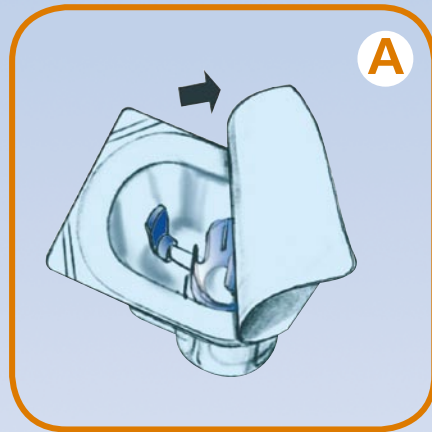

A

- Remove lid from package without touching the device

Do not remove device from package.

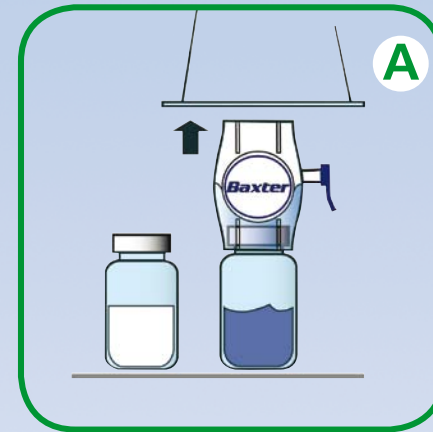


A

- Grip package at the edge and remove from device

Do not remove blue cap. Do not touch white spike.


Set up on hard surface

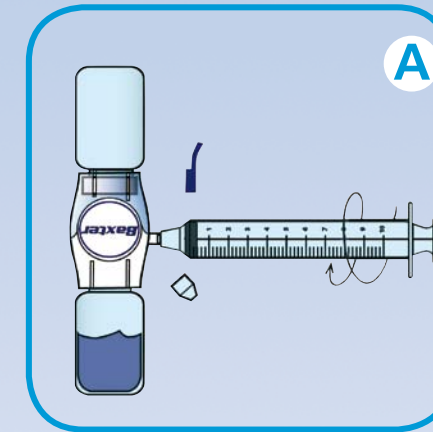

A

- Remove blue cap from device
- Remove syringe cap, if present
- Connect syringe to device


Do not inject air.

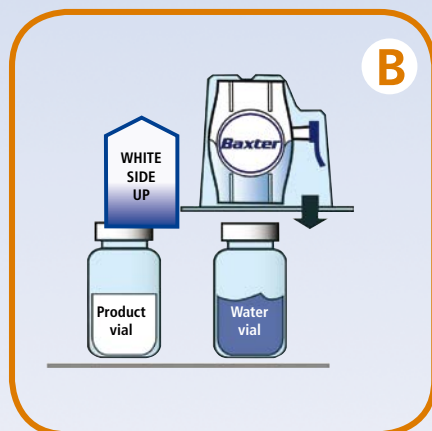

B

- Place vials on hard flat surface
- Invert device over water vial (WHITE SIDE UP)
- Press down until CLEAR spike pierces water vial stopper


Set up on hard surface

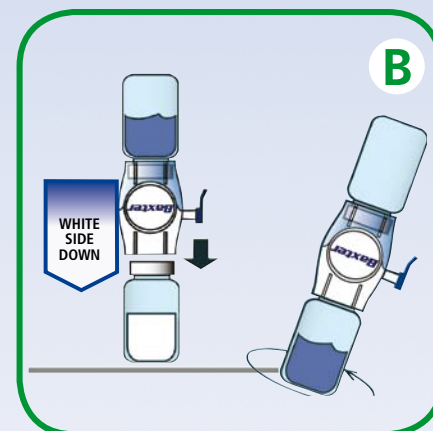

B

- Invert device with water vial attached (WHITE SIDE DOWN, water vial on top)
- Press down until WHITE spike pierces product vial stopper
- Wait for water to enter product vial
- Swirl gently to dissolve product

Set up on hard surface

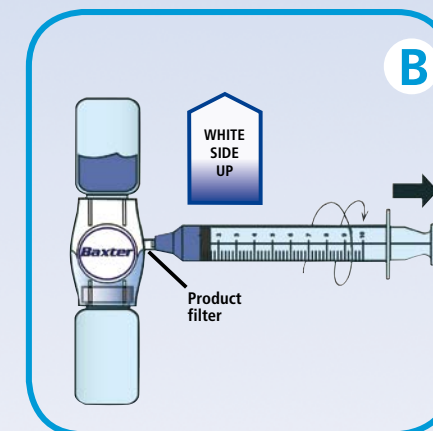

B

- Invert device (WHITE SIDE UP, product vial on top)
- SLOWLY pull plunger back to withdraw mixed product
- Disconnect syringe