

RCH Order of Draw - Tube Guide

S-Monovette® – The enclosed blood collection system


Please Note: Specimens for coagulation testing and Blood Cultures should be collected by Venepuncture and not Capillary Collection.

Blood Culture Refer to Specimen Reception Collection Manual

Sodium Citrate *MUST* be filled to the line, e.g. Coagulation Profile, INR, Prothrombin time




Serum Gel e.g. UEC, LFT, TFT, Ca, Magnesium, Phosphate, Amylase, Hormone profile, Viral Serology

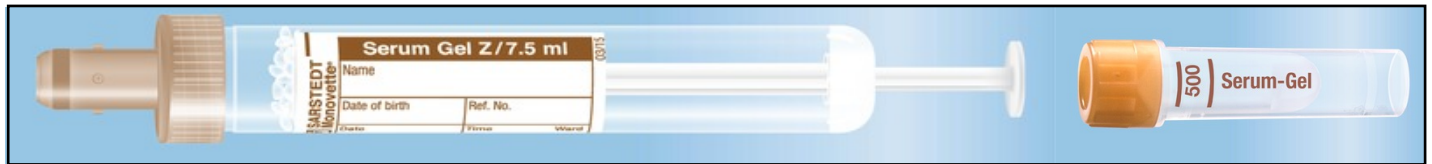

Lithium Heparin e.g. Chromosomes, Troponin, Amino Acids, Ammonia, Vit C, Zinc, TMPT



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
CPDA/ACD S-Monovette and Vacuum Tube. e.g. HLA Studies



EDTA e.g. FBE, Blood Group, X Match, HbA1c, PCR testing, DNA Studies



Fluoride Oxalate e.g. Glucose, Alcohol, Lactate

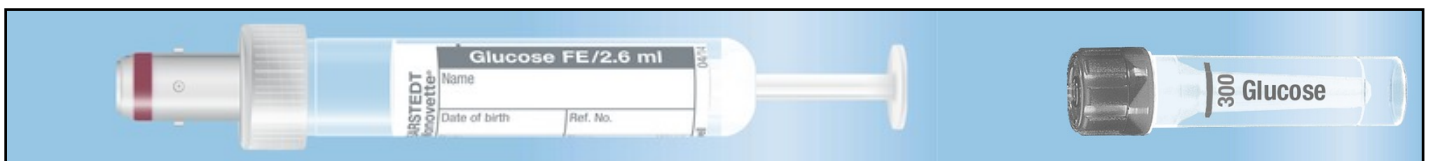

Specimen Collection Manual

Please consult Specimen Collection Manual for any further test requirements



Paediatric Blood Volume Guide

