

ATOM V-808

Cleaning Instructions



Parker Healthcare
www.parkerhealth.com.au

Parker Healthcare Pty Ltd

Head Office & International Department:

16 Redland Drive, Mitcham VIC 3132 Australia

Ph: +61 3 9872 0222 Fax: +61 3 9873 5911


Email: sales@parkerhealth.com.au

2014 Rev. 1.#

IMPORTANT

**Please ensure
all steps of the
cleaning regime
are followed
after each use.**

**PLEASE FOLLOW HOSPITAL PROTOCOL
FOR LINEN PROCESSING.**

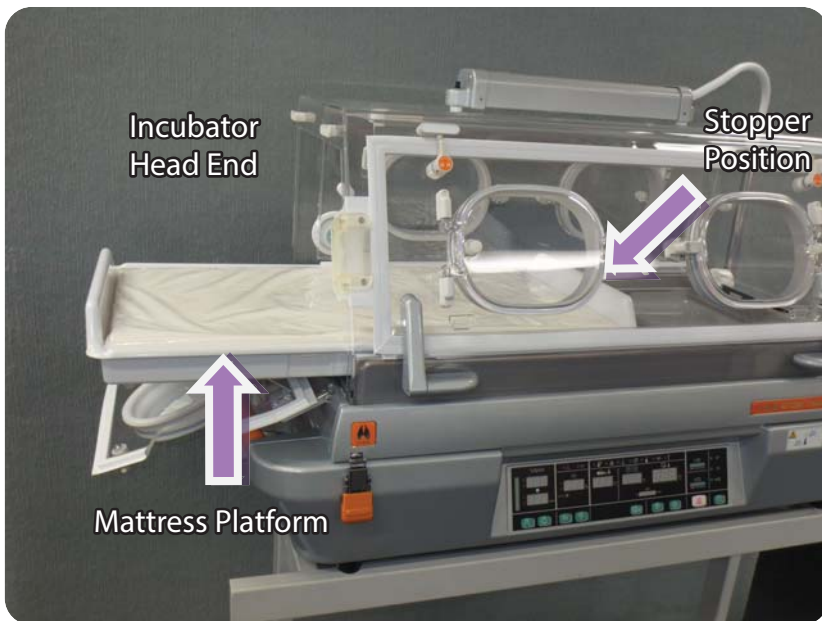

STEP 1

LINEN

Strip the linen from the incubator and process as per hospital protocol .

STEP 2

MATTRESS

Remove the mattress and wipe over with a damp cloth of warm water & hospital grade detergent.

DO NOT fully emerse mattress in cleaning solution.

PLATFORM

Open the head end admittance panel located on the left side of the hood, draw out the mattress platform toward you until the stopper is engaged.

To fully remove the mattress platform, press the stopper to disengage the lock and remove.

Wipe the mattress platform with a soft cloth moistened with hospital grade detergent.


STEP 3

STEP 4

ACCESS PORT


Remove the iris access port cover.

Immerse and clean in a hospital grade detergent.

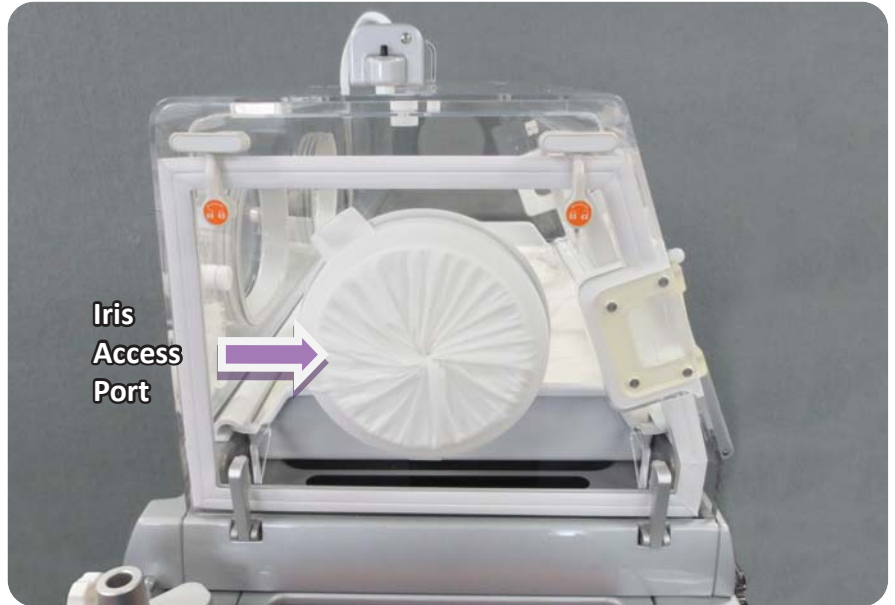

SNAP-OPEN ACCESS PORT PACKING

Remove the rubber packing from the snap-open access ports. Immerse and clean in a hospital grade detergent.

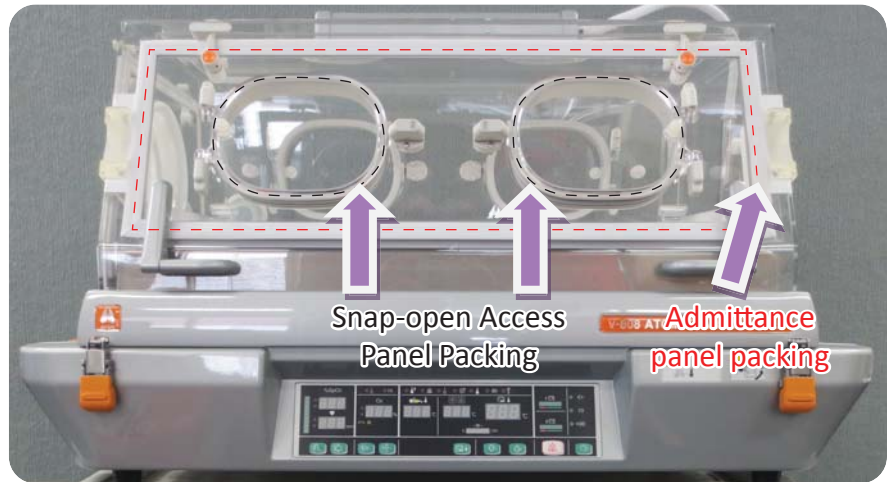

ADMITTANCE PANEL PACKING

Remove the rubber packing from the admittance panels. Immerse and clean in a hospital grade detergent.



STEP 5

STEP 6

TUBE INLET

Remove the packing from the tube inlets. (both sides of incubator) Immerse and clean in a hospital grade detergent.


**INSIDE PANEL
(FRONT, REAR,
TOP, RIGHT)**

1. Open the front admittance panel. Loosen the fixing screws for each inside panel (i.e. front, rear, top and right) and remove the inner panels.

2. Clean both sides of each inside panel with a soft cloth moistened with a hospital grade detergent.

**REMOVING
THE LIGHT**

Gently press the light release buttons on both sides of the light housing and lift in an upward direction.

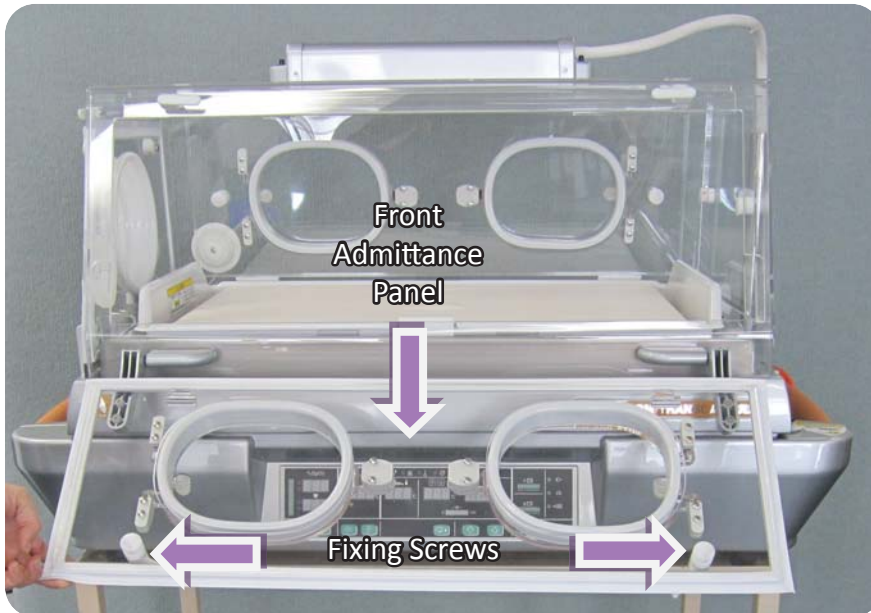

REMOVING THE HOOD

NOTE: THE FOLLOWING OPERATION WILL REQUIRE TWO (2) PEOPLE.


1. Unclip the front metal side clips (L & R) to release the hood from the base.

2. Pull the release screw at the back of the hood on both sides of the incubator.

NOTE: Removal of the hood will require two (2) people to complete.

Using one person on each side of the hood, lift up evenly and parallel to the base until the hood has been cleared from the incubator body.


Place on a flat surface and wipe over with a soft cloth dampened with hospital grade detergent.

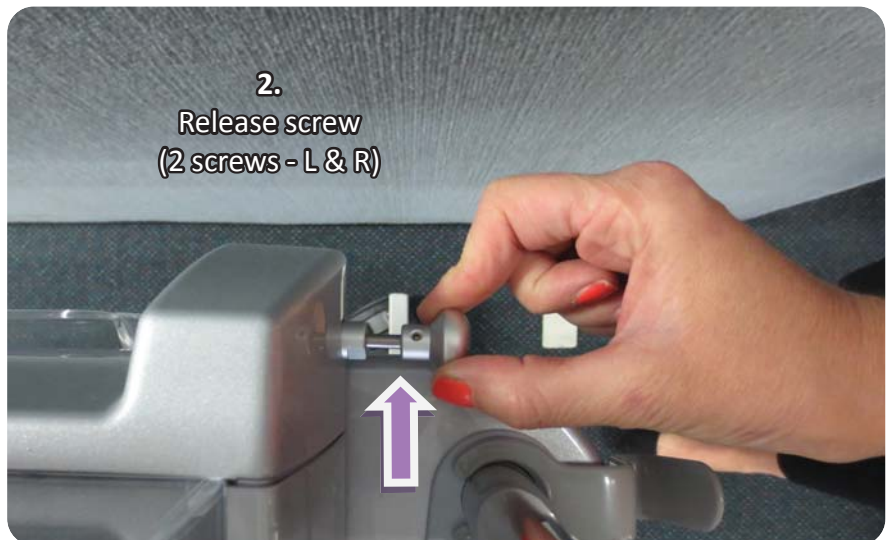

MIDDLE DECK PLATFORM

Hold both sides of the middle deck and lift.

Wipe over middle deck platform with a soft cloth moistened with hospital grade detergent

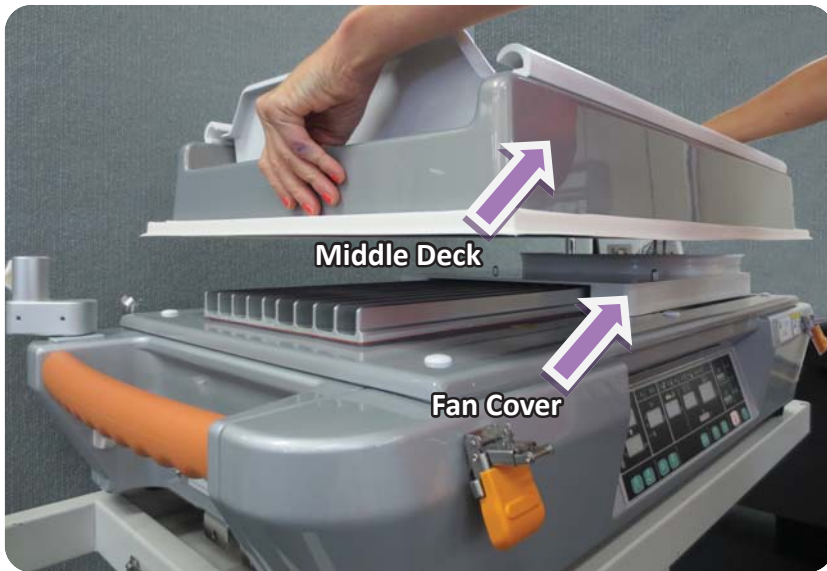


FAN COVER & FAN REMOVAL

Hold both sides of the fan cover and lift.

After removal of the fan cover, clean it with a soft cloth moistened with a hospital grade detergent.

To remove the fan, turn the fixing screw clockwise. Once the fixing screw has been removed, use 2 hands to lift the fan.

Immerse and clean in a hospital grade detergent.

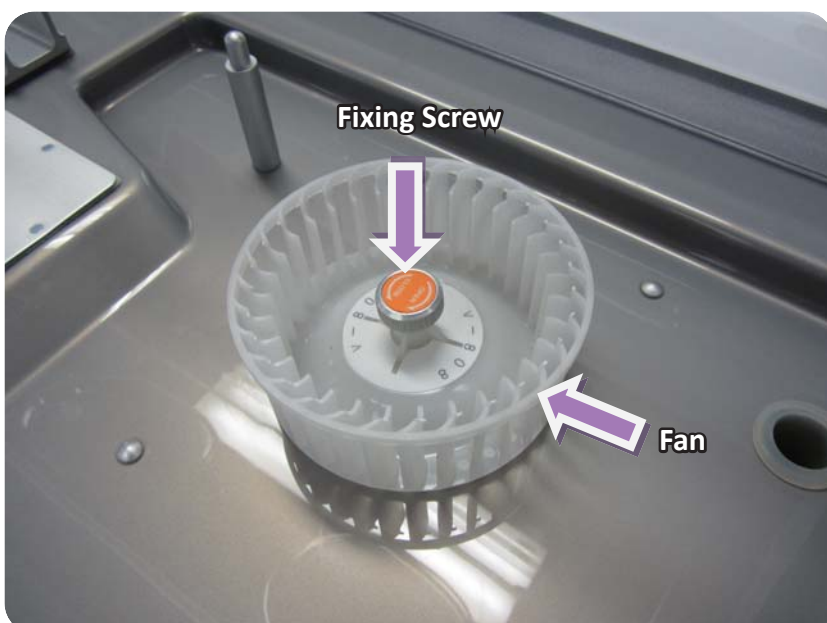

STEP 12

CONDITIONING CHAMBER

The conditioning chamber will appear when all the components mentioned previously have been removed.

Clean thoroughly with a soft cloth moistened with hospital grade detergent.

Ensure all parts are completely dry before undertaking re-assembly.

Re-assemble all removed parts in the reverse order of dis-assembly.

Check all parts are attached correctly.

Once re-assembled and checked, please ensure the incubator is plugged in, switched ON and the incubator temperature is set to 33 degrees.



STEP 13

SKIN TEMPERATURE PROBE

Wipe the entire probe (including thermostat) with a soft cloth moistened with hospital grade detergent between use.



This page is intentionally left blank.

Distributed by:

Parker Healthcare
www.parkerhealth.com.au

Parker Healthcare Pty Ltd

Head Office & International Department:

16 Redland Drive, Mitcham VIC 3132 Australia

Ph: +61 3 9872 0222 Fax: +61 3 9873 5911

Email: sales@parkerhealth.com.au