

How to set up humidity in an incubator

1. Collect the sterile humidity chamber (A) and the heater component (B)




A




B

2. Assemble the unit together

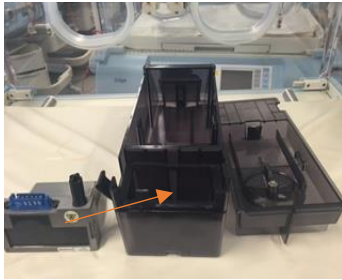

Heater unit fits into here

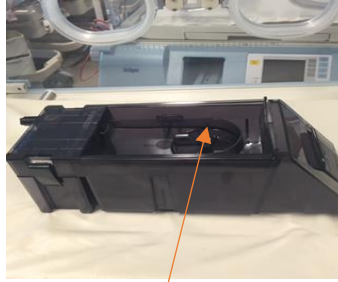

sterile water goes into here

3. Place the assembled unit into the incubator



Slide unit into space until secure



4. Turn on humidity by pressing humidity button (A) and then ON (B), dial in desired humidity (C)



A



B



C

When humidity is no longer needed the plastic humidity chamber is sent to CSSD, take care removing the heater unit as it may still be hot- this is not sent to CSSD, it is processed by the ward PSA's