



Exporting data

1. Open the EasyOne Connect Software Icon on your Desktop and go to Patients Tab
2. Select relevant Patient
3. Go to bottom of page and click on >> symbol

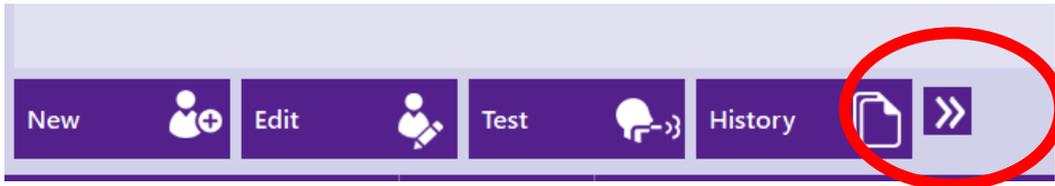


4. Click on **Database**
5. Choose Save File As and choose a location on the computer to save eg: Desktop or USB
6. Click **SAVE**
7. Then select **Export**

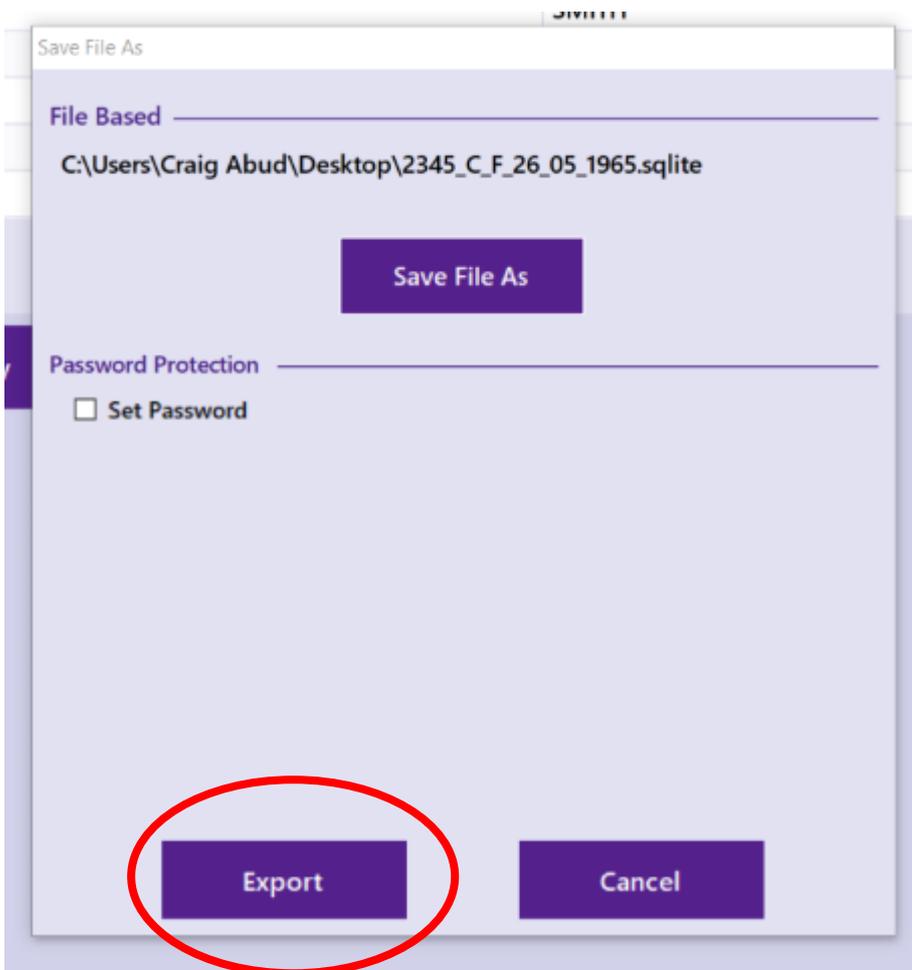


8. The Patient data is now exported to the chosen Computer location.

Importing data

1. Open the EasyOne Connect Software Icon on your Computer desk-top.
2. Go to: Utilities, select Configuration

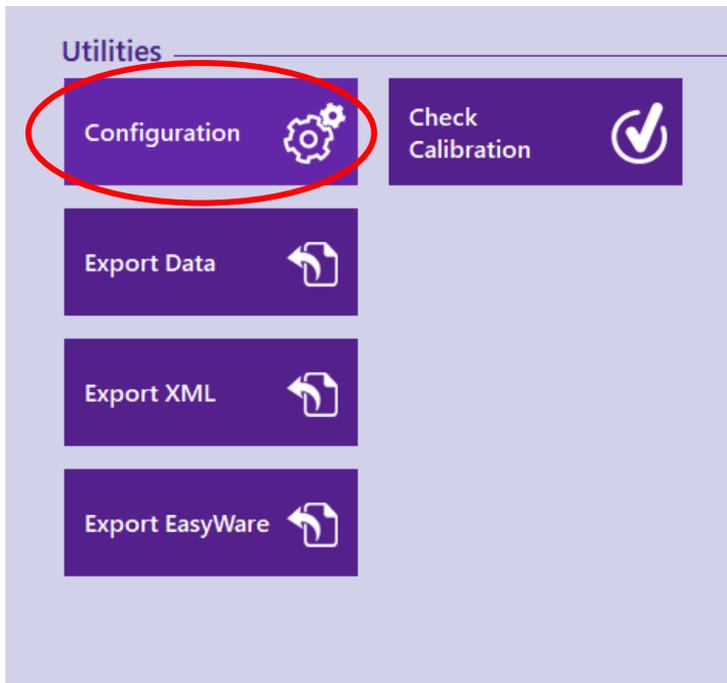

3. Select General –then Storage – then Import

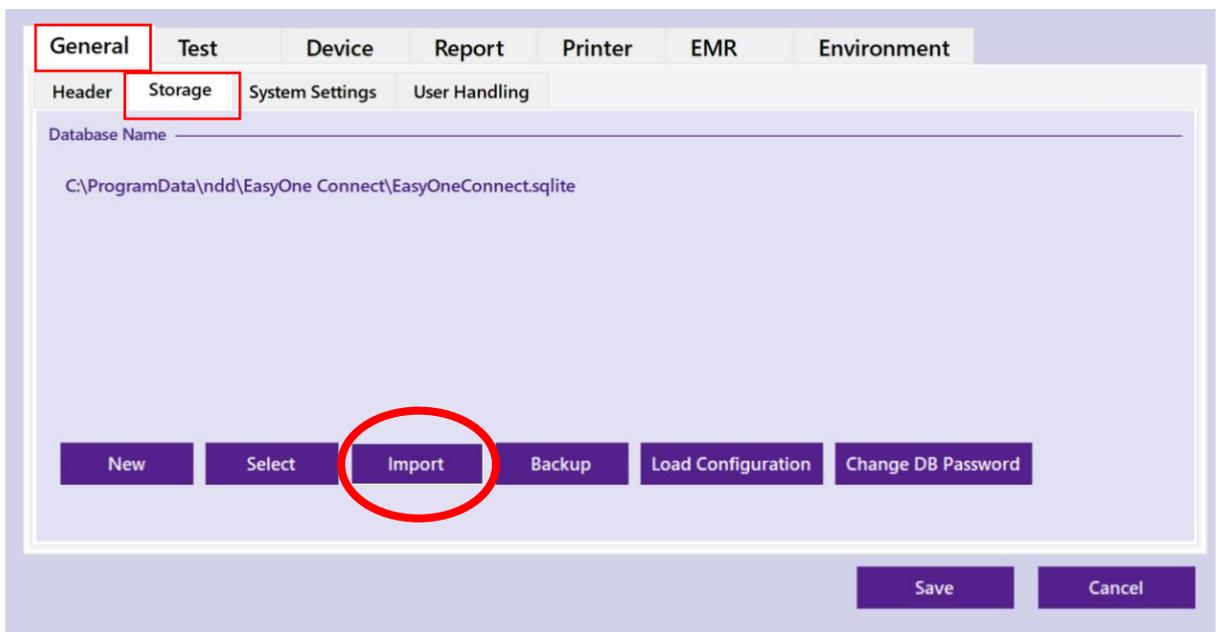

4. Select – Choose the Patient file from where you have saved it on your Computer or on a USB

5. Select OK – then Save

Select

Database Type

File based SQL Server based

File Based

Database Name

C:\Users\Craig Abud\Desktop\2345_C_F_26_05_1965.sqlite

Select

OK Cancel